



**::: Bones**

**Milestone**

**112 Ace**

**Addendum**

**www.bones.ch**

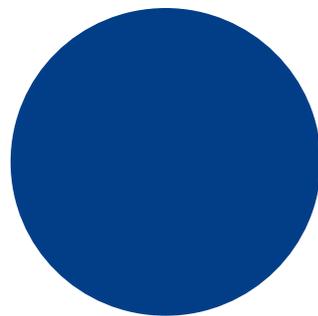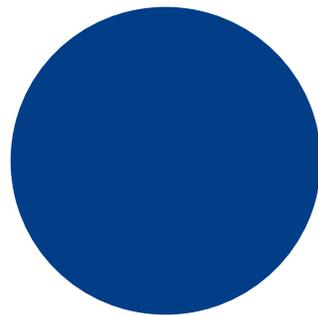

# 1. Clock

This chapter describes the usage of the clock. The clock is an option, which can be ordered at the purchase of the Milestone 112 Ace or later as well. If your M112 Ace does not have a clock and you would like to have one, please contact your dealer.

## 1.1. Check the time

To check the clock, please keep the selector key pressed for a short while, then release it again. The time you need to keep the key pressed is simple to remember. Just slowly speak the word „Milestone“ silently in your mind. It's about that long. This way, with the release of the button, the Milestone 112 Ace speaks clearly the current time in hours and minutes.

The time you need to keep the key pressed is necessary to omit accidental activation of the clock, when you select a folder.

## 1.2. Set the clock

To set the clock, keep the selector key pressed for at least 4 seconds. After this time, the Milestone 112 Ace says „Set the hour“ and speaks the current time. Now you can change the hour by using the left and right buttons. With the left button, you reduce the hour by one, with the right button, you increase the hour by one. After each change, the milestone tells you the time set. When you are done with the hours, press the play key to move on to the minutes. Now you can set the minutes in the same way as the hours. If you are done with the minutes, press the play key again. You have finished the time setting and the milestone speaks the set time.

## **2. Additional changes**

### **2.1. Startup**

The Milestone 112 Ace can be woken up from power saving mode with any key on the front. The selector key does not start the Milestone to avoid accidental startups in a bag for example.

### **2.2. Instant recording**

With the Milestone 112 Ace it is possible to start a recording at any time. It does not matter, if it is already running or still in power saving mode. Just press the recording button and keep it pressed, until you hear the recording click signal. Continue to keep the button pressed and make your recording after the click. Release the button to finish your recording.

### **2.3. Internal memory**

The internal memory has been increased to the size of 2.5 GB. This gives you space for about 80 hours of recordings.

### **2.4. Recording quality**

The recording quality has been improved. Recordings are now stored with 44.1 kHz.

### **2.5. Separate volume control**

The volume can now be set for the speaker and for the headphone separately. The Milestone 112 Ace automatically remembers each setting.

### **2.6. Battery life**

The hardware and software of the Milestone 112 Ace have been improved for better battery life. It is now able for up to 19 hours of continuous use.